

## Recommended protocol for CD4

Obtained in run 67

10 Jan 2023

### Immunostainer

Type: Leica BOND Max

### Primary antibody

Clone: IHC535  
Producer: GenomeMe  
Product no. / lot no.: IHC535 / C53522111  
Diluent: GenomeMe Antibody Diluent  
Diluent producer / no.: GenomeMe / IHC000  
Dilution factor: 1:200  
Incubation time / temperature: 15 min. / 20°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.