

Recommended protocol for URO II/III

Obtained in run 68

16 Feb 2023

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: BC21
Producer: Biocare Medical
Product no. / lot no.: ACI 3051 A, C / 011718
Diluent: Renoir Red Diluent
Diluent producer / no.: Biocare Medical / PD904
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 24 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 36°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 36°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.