

Recommended protocol for URO II/III

Obtained in run 68

16 Feb 2023

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: Uroplakin II BC21

Producer: Zytomed

Product no. / lot no.: MSG102 / A3991

Diluent: None

Diluent producer / no.: None / None

Dilution factor: 1:0

Incubation time / temperature: 60 min. / 37°C

Epitope retrieval, HIER

Device: On Board / On Machine

Buffer: Ventana CC1

Heating time at max. temp.: 64 min.

Maximum heating temp.: 95°C

Visualization system

Producer: Ventana

Product / no: UltraView Universal DAB Detection Kit / 760-500

Incubation time polymer: 8 min.

Incubation temperature: 36°C

Chromogen

Producer: Ventana

Product / no: ultraView Universal DAB Detection Kit / 760-500

Incubation time / temperature: 8 min. / 36°C

Enhancement: CuSO₄**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.