

Recommended protocol for MSH2

Obtained in run 68

17 Feb 2023

Immunostainer

Type: CNT330

Primary antibody

Clone: C2G3
Producer: Celnovte
Product no. / lot no.: CMM-0191 / 202301001
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Celnovte
Product / no: MicroStacker Flex Polymer Detection kit / SD5300
Linker: MicroStacker Flex Polymer Detection kit
Incubation time linker: 20 min.
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Celnovte
Product / no: MicroStacker Flex Polymer Detection kit / SD5300
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.