

Recommended protocol for PAX8

Obtained in run 68

24 Feb 2023

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: SP348
Producer: Abcam
Product no. / lot no.: ab227707 / GR3376680-12
Diluent: Renoir Red Diluent
Diluent producer / no.: Biocare Medical / PD904
Dilution factor: 1:100
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Amplifier: OptiView Amplification Kit
Incubation time amplifier: 4+4 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



Nordic Immunohistochemical Quality Control

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

the most recent developments in immunohistochemistry at any point in time.