

Recommended protocol for URO II/III

Obtained in run 68

24 Mar 2023

Immunostainer

Type: intelliPATH (Biocare)

Primary antibody

Clone: BC21 + BC17
Producer: Biocare Medical
Product no. / lot no.: API 3094 AA / 102821A
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Other
Buffer: Zytomed Systems HIER Citrate pH6
Heating time at max. temp.: 7 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Zytomed Systems
Product / no: ZytoChem Plus (HRP) Polymer Kit / POLHRP100
Linker: Postblock
Incubation time linker: 20 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Zytomed Systems
Product / no: DAB High Contrast / DABPLUS-5000
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.