

## Recommended protocol for CK8/18

Obtained in run 69

04 Jul 2023

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: EP17/EP30  
Producer: Dako  
Product no. / lot no.: M3652 / 11120072  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:50  
Incubation time / temperature: 32 min. / 37°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 96°C

**Visualization system**

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Incubation temperature: 37°C

**Chromogen**

Producer: Ventana  
Product / no: ultraView Universal DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 37°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.