

## Recommended protocol for CK8/18

Obtained in run 69

28 Sep 2023

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: EP17/EP30  
Producer: Dako  
Product no. / lot no.: M3652 / 11566443  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 20°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.