

## Recommended protocol for CK8/18

Obtained in run 69

16 Aug 2023

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: 5D3  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-5D3 / 6089831  
Diluent: Da Vinci Green  
Diluent producer / no.: Biocare Medical / PD900M  
Dilution factor: 1:100  
Incubation time / temperature: 40 min. / 20°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.