

## Recommended protocol for CK8/18

Obtained in run 69

16 Aug 2023

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: EP17/EP30  
Producer: Dako  
Product no. / lot no.: M3652 / 11343983  
Diluent: Bond Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 20°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Leica  
Product / no: Bond Polymer Refine Red Detection / DS9390  
Incubation time linker: 20 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Leica  
Product / no: Bond Polymer Refine Red Detection / DS9390  
Incubation time / temperature: 15 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.