

## Recommended protocol for PSA

Obtained in run 69

18 Aug 2023

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: EP109  
Producer: Biocare Medical  
Product no. / lot no.: CME 390 AK, CK / 070717  
Diluent: Renoir Red Diluent  
Diluent producer / no.: Biocare Medical / PD904  
Dilution factor: 1:100  
Incubation time / temperature: 15 min. / 22°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 1  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 22°C

**Chromogen**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.