

## Recommended protocol for CD34

Obtained in run

06 Jun 2016

**Immunostainer**

Type: IntelliPath FLX, Biocare

**Primary antibody**

Clone: QBend10  
Producer: Dako  
Product no. / lot no.: M7165 / 20023391  
Diluent: Da Vinci green  
Diluent producer / no.: Biocare / NS  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: Pressure Cooker  
Buffer: Biocare Diva Decloaker pH 6,2  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 110°C

**Visualization system**

Producer: Biocare  
Product / no: MACH 4 / NS  
Linker: MACH 4  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Biocare  
Product / no: DAB MACH 4 / NS  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.