

Recommended protocol for ER

Obtained in run

19 Jun 2016

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: 6F11
Producer: Leica/novocastra
Product no. / lot no.: NCL-ER-6F11 / 6037182
Diluent: Antibody Diluent
Dilution factor: 1:25
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 30 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Dako
Product / no: EnVision+, Mouse / K4000/K4001
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.