

Recommended protocol for ERG

Obtained in run 50

07 Mar 2017

Immunostainer

Type: Ventana Benchmark XT

Primary antibody

Clone: EP111
Producer: Dako
Product no. / lot no.: M7314 / 20023875
Diluent: Antibody Dilution Buffer
Dilution factor: 1:50
Incubation time / temperature: 32 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 32 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 37°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.