

Recommended protocol for CK19

Obtained in run 50

07 Mar 2017

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: BS23
Producer: Biosite
Product no. / lot no.: BSH-7240-1 / BSH1002cth
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:250
Incubation time / temperature: 30 min. / 30°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 24 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 30°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 10 min. / 30°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.