

## Recommended protocol for Bcl-6

Obtained in run 70

18 Dec 2023

**Immunostainer**

Type: Gene Tech GeneStainer

**Primary antibody**

Clone: LN22  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-Bcl-6-564 / 6082532  
Diluent: Ab diluent  
Diluent producer / no.: Gene Tech / GT100910  
Dilution factor: 1:50  
Incubation time / temperature: 50 min. / 25°C

**Epitope retrieval, HIER**

Device: Water bath  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: Gene Tech  
Product / no: GTVision / GK8007  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

**Chromogen**

Producer: Gene Tech  
Product / no: DAB / GK8007  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.