

## Recommended protocol for Bcl-6

Obtained in run 70

09 Jan 2024

**Immunostainer**

Type: Thermo Autostainer 36/48/72

**Primary antibody**

Clone: LN22  
Producer: Epredia  
Product no. / lot no.: MS-1936-S / 1936P0523  
Diluent: Bright diluent  
Diluent producer / no.: Immunologic / UD09-999  
Dilution factor: 1:100  
Incubation time / temperature: 60 min. / 25°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

**Visualization system**

Producer: Immunologic  
Product / no: Brightvision / UBVB999HRP-polymer  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 25°C

**Chromogen**

Producer: Immunologic  
Product / no: Bright DAB / UBVB999HRP-BSO4B  
Incubation time / temperature: 8 min. / 25°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.