

## Recommended protocol for PMS2

Obtained in run 71

16 Feb 2024

**Immunostainer**

Type: Leica BOND III

**Primary antibody**

Clone: EP51  
Producer: Cell Marque  
Product no. / lot no.: AC-0049 / 230100790  
Diluent: Diamond - Antibody Diluent  
Diluent producer / no.: Cell Marque / 938B  
Dilution factor: 1:50  
Incubation time / temperature: 15 min. / 22°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 100°C

**Visualization system**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 21°C

**Chromogen**

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.