

## Recommended protocol for PMS2

Obtained in run 71

19 Feb 2024

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: EP51  
Producer: Dako  
Product no. / lot no.: M3647 / 11593174  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 24°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 24°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 24°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.