

Recommended protocol for CD117

Obtained in run 71

27 Mar 2024

Immunostainer

Type: Leica Bond Prime

Primary antibody

Clone: EP10
Producer: Leica Biosystems
Product no. / lot no.: PA0007 / 77497
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 15 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Leica Bond Prime Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 104°C

Visualization system

Producer: Leica
Product / no: Bond Prime Polymer DAB Detection System / DS9284
Linker: None
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Prime Polymer DAB Detection System / DS9284
Incubation time / temperature: 8 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.