

## Recommended protocol for p16

Obtained in run 72

08 Jul 2024

### Immunostainer

Type: Leica BOND III

### Primary antibody

Clone: MX007  
Producer: vitro master diagnostic ITK  
Product no. / lot no.: MAD-000690Q / 0690Q134  
Diluent: Renoir Red Diluent  
Diluent producer / no.: Biocare Medical / PD904  
Dilution factor: 1:100  
Incubation time / temperature: 15 min. / 21°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica BOND-PRIME Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.