

Recommended protocol for Ki67

Obtained in run 72

10 Jul 2024

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: Mib1
Producer: Dako
Product no. / lot no.: IR626 / 41693089
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 20 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS Low pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 20 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.