

## Recommended protocol for CK19

Obtained in run 50

17 Mar 2017

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: BA17  
Producer: NeoMarkers  
Product no. / lot no.: MS-1671-P1 / 1671P1605C  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:100  
Incubation time / temperature: 16 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 48 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.