

## Recommended protocol for TRPS1

Obtained in run 72

10 Oct 2024

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: EP392  
Producer: BIO SB  
Product no. / lot no.: BSB-3790-1 / 3790PGOB12  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:100  
Incubation time / temperature: 20 min. / 20°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.