

Recommended protocol for Ki67

Obtained in run 72

30 Aug 2024

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: SP6
Producer: Master Diagnostica
Product no. / lot no.: MAD-00031Q / 0310Q151
Diluent: REAL Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:100
Incubation time / temperature: 20 min. / 35°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 35°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 35°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.