

## Recommended protocol for Bcl-2

Obtained in run 73

12 Dec 2024

**Immunostainer**

Type: Titan

**Primary antibody**

Clone: MX022  
Producer: Fuzhou Maixin  
Product no. / lot no.: MAB-0711 / 2411060711d  
Diluent: None  
Diluent producer / no.: (None) / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 26°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Fuzhou Maixin DNS  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

**Visualization system**

Producer: Fuzhou Maixin  
Product / no: Titan Super / TT-0805  
Linker: Super  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 26°C

**Chromogen**

Producer: Fuzhou Maixin  
Product / no: DAB / TT-0805  
Incubation time / temperature: 5 min. / 26°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.