

## Recommended protocol for CD30

Obtained in run 74

24 Apr 2025

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: Ber-H2  
Producer: Agilent  
Product no. / lot no.: GA602 / 41654197  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 10 min. / 32°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, Low pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823/GV900  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.