

## Recommended protocol for MLA

Obtained in run 74

12 Feb 2025

**Immunostainer**

Type: Ventana Benchmark GX

**Primary antibody**

Clone: A103  
Producer: Dako  
Product no. / lot no.: M7196 / 20017531  
Diluent: REAL Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:25  
Incubation time / temperature: 30 min. / 37°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 98°C

**Visualization system**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 4 min.  
Incubation time polymer: 4 min.  
Amplifier: OptiView Amplification Kit  
Incubation time amplifier: 4+4 min.  
Incubation temperature: 37°C

**Chromogen**

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 37°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



## **Nordic Immunohistochemical Quality Control**

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

the most recent developments in immunohistochemistry at any point in time.