

Recommended protocol for BRAF

Obtained in run 74

17 Mar 2025

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: VE1
Producer: Abcam
Product no. / lot no.: ab228461 / GR3427211-5
Diluent: Antibody Diluent with Background Reducing Component
Diluent producer / no.: Dako / S3022
Dilution factor: 1:200
Incubation time / temperature: 20 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 60 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.