

Recommended protocol for MLA

Obtained in run 74

03 Apr 2025

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: BS52
Producer: Nordic Biosite
Product no. / lot no.: BSH-2003-1 / BSH1-18ac
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 24°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 30 min.
Incubation temperature: 24°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 24°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.