

## Recommended protocol for MLA

Obtained in run 74

04 Apr 2025

**Immunostainer**

Type: Sakura Tissue-Tek Genie

**Primary antibody**

Clone: EP43  
Producer: Sakura FineTek  
Product no. / lot no.: 8319-C010 / 8319E04125  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 30 min. / 20°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Sakura Tissue-Tek Genie High pH Antigen Retrieval  
Heating time at max. temp.: 60 min.  
Maximum heating temp.: 98°C

**Visualization system**

Producer: Sakura  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Sakura  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.