

Recommended protocol for HER2 IHC

Obtained in run B40

04 Jun 2025

Immunostainer

Type: Ventana Benchmark Ultra Plus

Primary antibody

Clone: PATHWAY (clone 4B5)
Producer: Ventana
Product no. / lot no.: 790-2991 / M22842
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 12 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 36 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.