

Recommended protocol for SOX10

Obtained in run 75

30 Sep 2025

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: BS7
Producer: Nordic Biosite
Product no. / lot no.: BSH-7959-XXX / bsh136aj
Diluent: REAL Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:200
Incubation time / temperature: 20 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.