

## Recommended protocol for HER2 IHC

Obtained in run B40

25 Sep 2025

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: HercepTest (polyclonal)  
Producer: Dako  
Product no. / lot no.: SK001 / 41846238  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 30 min. / 21°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako Herceptest  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Dako  
Product / no: Herceptest / SK001  
Incubation time polymer: 30 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Dako  
Product / no: Herceptest / SK001  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.