

## Recommended protocol for SOX10

Obtained in run 75

25 Jun 2025

**Immunostainer**

Type: Dako Omnis

**Primary antibody**

Clone: BC34  
Producer: Biocare Medical  
Product no. / lot no.: AC13099A or C / 120123AZ  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 32°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823/GV900  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 32°C

**Chromogen**

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.