

Recommended protocol for AMACR

Obtained in run 75

14 Aug 2025

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: 13H4
Producer: Dako
Product no. / lot no.: M3616 / 11580030
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Water bath
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 25 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.