

Recommended protocol for HER2 IHC

Obtained in run B40

27 Aug 2025

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: 4B5
Producer: Ventana Roche
Product no. / lot no.: 790-4493 / M26878
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 12 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 36 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 4 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 4 min. / 36°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.