

## Recommended protocol for CDX2

Obtained in run 76

05 Dec 2025

**Immunostainer**

Type: Ventana Benchmark Ultra

**Primary antibody**

Clone: EPR2764Y  
Producer: Cell Marque  
Product no. / lot no.: 235R-14, -15 or -16 / 0000214092  
Diluent: Antibody Diluent  
Diluent producer / no.: Ventana / 251-018  
Dilution factor: 1:200  
Incubation time / temperature: 32 min. / 36°C

**Epitope retrieval, HIER**

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

**Visualization system**

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Amplifier: Ultraview/iView Amplification Kit  
Incubation time amplifier: 8+8 min.  
Incubation temperature: 36°C

**Chromogen**

Producer: Ventana  
Product / no: ultraView Universal DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.