

Recommended protocol for ER

Obtained in run B24

14 Aug 2017

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: SP1
Producer: Spring Biosystems
Product no. / lot no.: M3010/M3012/M3014 / 150108LVK
Diluent: Antibody Diluent
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Nordic Biosite
Product / no: BioSite Histo Plus HRP Polymer anti-Rabbit / KDB-10046
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: Nordic Biosite
Product / no: High contrast DAB / BCB-20032
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.