

Recommended protocol for NKX3.1

Obtained in run 77

19 Mar 2026

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: EP356
Producer: Cell Marque
Product no. / lot no.: 441R-14/441R-15/441R / 0000344747
Diluent: Diamond - Antibody Diluent
Diluent producer / no.: Cell Marque / 938B
Dilution factor: 1:25
Incubation time / temperature: 15 min. / 22°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 30 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 21°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.