

## Recommended protocol for GLP3

Obtained in run

13 Sep 2015

**Immunostainer**

Type: Dako Autostainer Link 48 +

**Primary antibody**

Clone: 1G12  
Producer: Cell Marque  
Product no. / lot no.: 261M-95 / 1200610B  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S3022  
Dilution factor: 1:200  
Incubation time / temperature: 20 min. / 20°C

**Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Dako TRS High pH  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

**Visualization system**

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

**Chromogen**

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

**Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.